



How to Give Your Pet Transdermal Medication

Follow instructions on the prescription label before use.

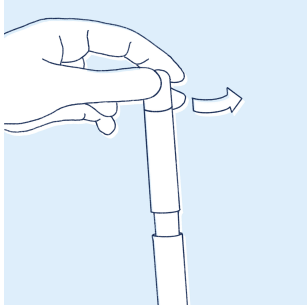


Step 1

Unscrew the cap. Put aside for recapping after use.

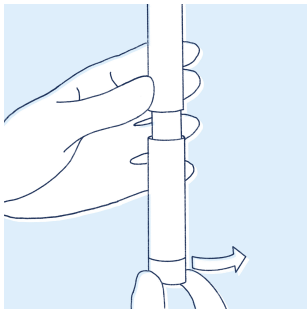


Step 2

Hold the applicator in an upright position, then turn (click) the base the number of mLs shown on the prescription label. **Please Note:** The Audible, Tactile, and Visual CLICK points of the triad shaped base must be used as dosing indicators. Align the Base and Triad Tube to fully dispense each dose.

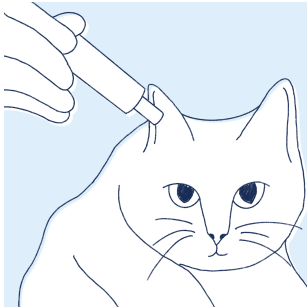

Step 3

Use the built-in applicator to apply medication as directed by your veterinarian or pharmacist. Rub continually until preparation has saturated the prescribed areas of your pet's fur. **Please Note:** The built-in applicator minimizes the chance of the medication spreading to unintended areas. It is not necessary to use your fingertips to apply this medication.

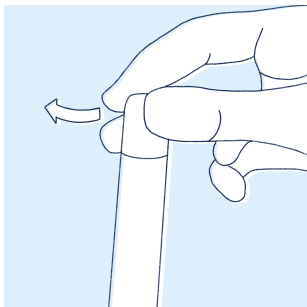

Step 4

To prevent evaporation, screw the cap back on when finished with each dosage. Be sure the cap is tightly secured.

The Refill Badge reminder system helps you reorder on time.

When the Plunger's TOP black line reaches the embossed line below "REFILL," there are approximately 32 CLICKS remaining. At the line above "REFILL," there are approximately 12 CLICKS remaining.

Top-CLICK Micro is not re-fillable.

Made of FDA grade materials. Phthlate free. Blocks UV and is recyclable.

DISCLAIMER Always wear gloves. Always consult your veterinarian.