

AlphaTrak3

BLOOD GLUCOSE MONITORING SYSTEM



Bluetooth® Setup Guide

1. Enable discovery mode on your meter



Press for less than 3 sec to turn on your meter.



Press again for less than 3 sec to view result history.




Press and hold both and together for 3 sec to toggle Bluetooth® discovery mode.


will appear blinking on the screen.

While meter is off, press and hold “c” for more than 3 sec to enable/disable automatic Bluetooth discovery with every result.


2. Pair with your mobile

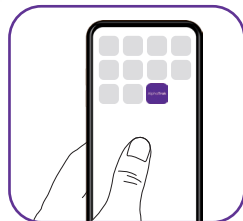


Open the **AlphaTrak** application.

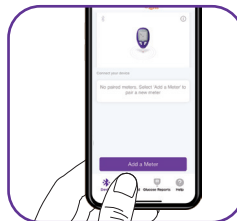

Tap **Device** at the bottom navigation. Tap **Add a Meter** to find your meter.

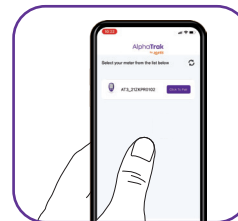

Tap if you don't automatically find your meter. Select your meter and tap **Click to Pair**.

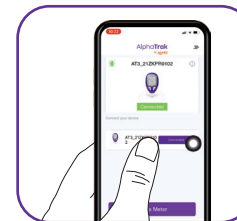

Enter the 6-digit pairing key, now displayed on your meter, to the application.

Application will confirm that you've successfully connected. On the meter, the Bluetooth® icon will stop blinking.

3. Import test data



Tap **Dashboard** at the bottom navigation.



Tap **Get Result** to get test results from your meter.



Application will confirm that test results successfully synced.

When getting test results, if the Bluetooth® mode is set to turn on automatically, you will see the blinking Bluetooth® icon once the result is ready. This indicates that Bluetooth® is on and ready to pair with the AlphaTrak App.

For additional information, please contact Zoetis Inc.
Customer Service: 1-888-963-8471
Customer Service for Canada: 1 800 461 0917
Zoetis, 333 Portage Street, Kalamazoo, MI 49007, USA

AlphaTrak.com

All trademarks are the property of Zoetis Services LLC or a related company or a licensor unless otherwise noted.
© 2023 Zoetis Services LLC. All rights reserved. ATK-00363

zoetis